



Helping all people live healthy lives

NOW BD ECLIPSE™ NEEDLES COME IN MORE SIZES AND OPTIONS THAN EVER BEFORE!

- A wide range of choices is available for both luer lock and luer slip applications.
- SmartSlip™ Technology, a new engineering and work practice control, helps ensure secure connections to luer slip syringes.
- As always, the BD Eclipse Family offers one safety technology for both your injection and blood collection needs.

Now, more than ever, BD Eclipse Needles are designed to meet the demands of today's healthcare settings.



3. Injection

Administer injection according to protocol.

For user convenience, the needle "bevel-up" position is oriented to safety cover.

(Bevel-up = Safety cover up)

4. Activation

Immediately following injection, activate safety cover by centering thumb or forefinger (of hand holding syringe) on finger pad area of safety cover, then pushing cover forward until you hear/feel it lock.

Visually confirm that the needle tip is covered.



* If twisting motion is used with Luer Slip syringes be sure to push and twist simultaneously for secure assembly

** If changing needles, activate safety cover prior to removing the needle from the syringe



CATALOG #	NEEDLE SIZE	PACKAGING
LUER LOCK		
BD Eclipse Needle for Luer Lock Syringes		
305761	25G x 1"	100/Box - 1,200/Case
305767	25G x 1 ½"	100/Box - 1,200/Case
305762	23G x 1"	100/Box - 1,200/Case
305768	22G x 1"	100/Box - 1,200/Case
305763	22G x 1 ½"	100/Box - 1,200/Case
305764	21G x 1"	100/Box - 1,200/Case
305765	21G x 1 ½"	100/Box - 1,200/Case
305766	18G x 1 ½"	100/Box - 1,200/Case

BD Eclipse Needle for Luer Lock Syringe -NEW SIZES!		
305759	25G x 5/8"	100/Box - 1,200/Case
305758	27G x ½"	100/Box - 1,200/Case
305757	30G x ½"	100/Box - 1,200/Case

BD Eclipse Needle with Luer Lock Syringe		
1mL BD Luer-Lok™ Syringe with Detachable Needle		
305778	30G x ½"	50/Box - 300 Case
305789	27G x ½"	50/Box - 300 Case
305780	25G x 5/8"	50/Box - 300 Case

3mL BD Luer-Lok™ Syringe with Detachable Needle		
305781	25G x 5/8"	50/Box - 300 Case
305787	25G x 1"	50/Box - 300 Case
305782	23G x 1"	50/Box - 300 Case
305788	22G x 1"	50/Box - 300 Case
305783	22G x 1 ½"	50/Box - 300 Case
305779	21G x 1"	50/Box - 300 Case
305784	21G x 1 ½"	50/Box - 300 Case

5mL BD Luer-Lok™ Syringe with Detachable Needle		
305785	22G x 1 ½"	50/Box - 300 Case

10mL BD Luer-Lok™ Syringe with Detachable Needle		
305786	22G x 1 ½"	50/Box - 300 Case

LUER SLIP		
BD Eclipse Needle with SmartSlip™ Technology		
305760	25G x 5/8"	100/Box - 1,200/Case
305770	27G x ½"	100/Box - 1,200/Case
305771	30G x ½"	100/Box - 1,200/Case

1mL Luer Slip Syringe with Detachable Needle-NEW!		
305775	30G x ½"	50/Box - 300/Case
305776	27G x ½"	50/Box - 300/Case
305777	25G x 5/8"	50/Box - 300/Case

Safety Within Your Reach

BD Eclipse™ Needle



How To Use BD Eclipse™ Needle and BD Eclipse™ Needle with SmartSlip™ Technology



1. Attachment

BD Eclipse™ Needles For Luer Lock Syringes
Firmly attach the needle to the Luer Lock syringe with a firm push and a clockwise twist.



BD Eclipse™ Needles With SmartSlip™ Technology for Luer Slip syringes
Attach the needle that features SmartSlip™ Technology by pushing onto the Luer Slip syringe firmly until an audible click is heard.*

2. Aspiration

Draw up medication in accordance with established protocol.**



- ### EASE OF USE
- Single-handed activation
 - No hard surface required for activation
 - Requires minimal change in injection technique

- ### DESIGNED FOR SAFETY
- Safety cover is present between clinician and needle
 - Audible click indicates safety activation
 - Needle remains locked inside activated cover

- ### A BROAD SELECTION
- Includes 18 gauge needle for filling and other "large bore" applications
 - Includes new SmartSlip™ Technology (on select small gauge needle sizes), which is designed for use with luer slip syringes

Safety Within Your Reach

