

Modern Way
Immobilizers,
Inc.
PO Box 660
104 Johnson Street
Clifton, TN 38425
Tel: 931.676.5274
Fax: 931.676.3860

Made in U.S.A.
U.S. Patent No. 5,600,702

[Our Home Page](#)

[Whats New](#)

[About Our Products](#)

[How To Operate](#)

[Our Guarantee](#)

[Order Online](#)

[Purchase Order Form](#)

[Replacement Parts](#)

[For Dealers Only](#)

[Establish A Line Of Credit](#)

[Contact Us](#)

[Site Map](#)

[Our Privacy Policy](#)


Need More
Product Information?
[Click Here](#)



About Our Products

Thursday, May 20, 2010

The Pigg-O-Stat Pediatric Immobilizer and Positioner is approximately 20" x 20" x 35" (50cm x 55cm x 88 cm) in height. Stainless steel table with 4" (7.5cm) casters, 2 are locking. Table top and shelf are made of high density polyethylene in child-pleasing primary colors which requires no maintenance and is virtually indestructible. Other moving parts are heavy plexiglass.

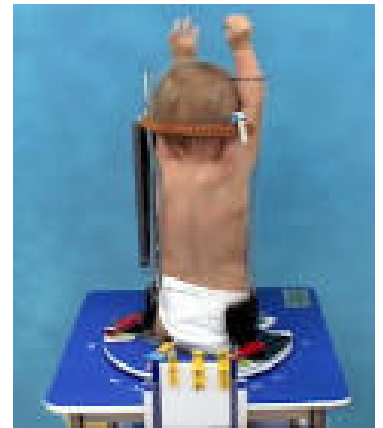


- ❖ Seat platform turns on bearings.
- ❖ Cassette carrier has adjustable horizontal and vertical movement for precise positioning of film.
- ❖ The scale in degree readings on table top ensures exact duplication of positions for re-check examinations.
- ❖ Adjustable seat in center of table top with openings for patients legs.
- ❖ Seat platform can be rotated horizontally to any degree and position.
- ❖ Form-fitting body, head and arm supports of transparent plexiglass are provided.
- ❖ Adjustable for patients from infants to 3-1/2 years of normal size for age.
- ❖ Lead shield has adjustable conventional markers and adjusts up and down for various examinations.
- ❖ Elimination of position and immobilizing problems makes it easier to make exposure at proper stage of respiration.
- ❖ Spring-balanced cassette holder accommodates any size cassette needed.
- ❖ Device is easily cleaned and sterilized.
- ❖ The Product will last for years and years with proper care.

Copyright © 1997 - 2003 Modern Way Immobilizers, Inc. All Rights Reserved
PO Box 660 104 Johnson Street Clifton, TN 38425
Tel: 931.676.5274 Fax: 931.676.3860 Toll Free: 866-MWI-PIGG (866-694-7444)

This site was built and is maintained by: [NetEase Internet Access Service](#)
For Questions regarding this site please contact our webmaster@netease.net

NetEase
Web Solutions
For Your Business

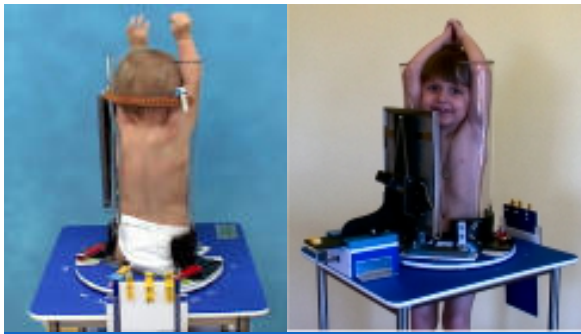

Modern Way
Immobilizers,
Inc.
PO Box 660
104 Johnson Street
Clifton, TN 38425
Tel: 931.676.5274
Fax: 931.676.3860

Made in U.S.A.
U.S. Patent No. 5,600,702

[Our Home Page](#)

[Whats New](#)

[About Our Products](#)

[How To Operate](#)

[Our Guarantee](#)

[Order Online](#)

[Purchase Order Form](#)

[Replacement Parts](#)

[For Dealers Only](#)

[Establish A Line Of Credit](#)

[Contact Us](#)

[Site Map](#)

[Our Privacy Policy](#)

Need More
Product Information?
[Click Here](#)



How To Operate

Thursday, May 20, 2010

Operation Instructions

1. As is true of any new equipment, you will find its operation simple and easy after a little experience. It is therefore suggested that your first few x-ray examinations be on children in the age group of 12 to 18 months using large supports.
2. First of all set technique controls, position tube and select proper film size, while all the child's clothes except diaper are being removed.
3. Open the supports on the immobilizer (as a book). Keep seat as low as possible and yet have child's mouth up to level of opening in front of supports.
4. Place child on the seat, instruct someone to hold child's arms in a vertical position touching ears and firmly immobilizing head between arms. Then adjust supports firmly against the child and fasten locks on bases and leather straps at top of supports. Re-adjust child if not in absolute erect position.
5. Always have patient well immobilized before permitting assistant to turn loose of arms and head.
6. Avoid using device for patients who are too large.
7. Adjust film to proper height and in contact with supports; also, adjust lead shield and markers.
8. For barium studies, place small plastic sponge under child's chin to prevent barium from spilling inside supports.
9. Your technical factors will be changed very little by use of supports.



Proper Care and Use of the Immobilizer & Positioner

- ❖ Even though this equipment is very durable and will withstand many times the required stress for proper use (as proven in extensive use and test) precaution, however, should be taken against hard blows or dropping the supports.

- ❖ Equipment may be sterilized with Zephiran Chloride.
- ❖ Plexiglass parts may be cleaned with soap and water using a soft sponge or cloth; rinse with clear water.
- ❖ A few drops of oil may be used on moving parts.

[Home](#) | [Our Products](#) | [What's New](#) | [How To Use](#) | [Replacement Parts](#) | [Dealers Only](#) | [Contact Us](#) | [Our Privacy Policy](#)

Copyright © 1997 - 2003 Modern Way Immobilizers, Inc. All Rights Reserved
PO Box 660 104 Johnson Street Clifton, TN 38425
Tel: 931.676.5274 Fax: 931.676.3860 Toll Free: 866-MWI-PIGG (866-694-7444)

This site was built and is maintained by: [NetEase Internet Access Service](#)
For Questions regarding this site please contact our webmaster@netease.net

NetEase
Web Solutions
For Your Business