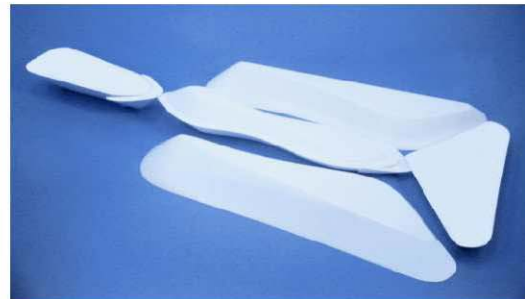
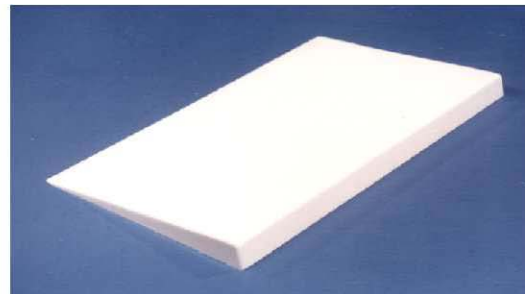
Les filtres

Filters

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for a more
homogenous
density

Jean A. Vézina M.D.

Collaboration

CLAIRE AYOTTE T.R.
FRANCINE LABRANCHE T.R.
GHISLAINE MORIN T.R.
Hôpital du Sacré-Coeur, Montréal
DENIS PROULX T.R.
Hôpital Notre-Dame, Montréal

Contributors

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JACQUES VÉZINA
PIERRE VÉZINA

Presentation
and Layout

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JEAN A. VÉZINA M.D.

Photography

COPYRIGHT 2001 OCTOSTOP[®] Inc.

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All rights reserved
FOR ALL COUNTRIES
Registered Trademark

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Technologists do
some miracles...
The art of filtering
lets them perform daily!

<i>NOTIONS de BASE</i>	BASIC NOTIONS	pages 2-4
<i>le BOOMERANG</i>	the BOOMERANG	5-8
<i>le LINGOT</i>	the INGOT	9-11
<i>la PENTE DOUCE</i>	the GENTLE SLOPE	12
<i>le PETIT PRISME</i>	the LITTLE PRISM	13
<i>les BORDURES PLUS</i>	the RIDGES PLUS	14
<i>le PHARAON</i>	the PHARAOH	15
<i>DIVERS</i>	MISCELLANEOUS	16
<i>PRÉCAUTIONS</i>	PRECAUTIONS	17

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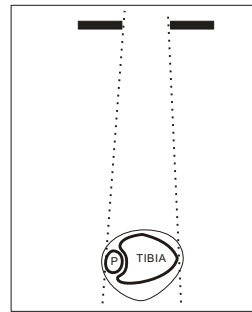
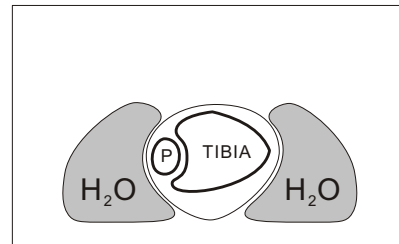
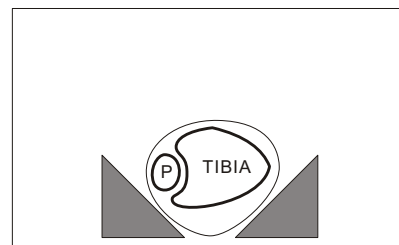
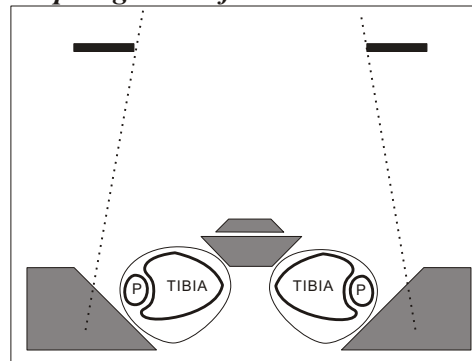
BASIC NOTIONS

OBJECTIVES

To even out the density of an image which would otherwise be too bright or too dark in some areas. This saves films, time and radiation. Also to immobilize at the same time.

MEANS

1. Closure of the **diaphragms** inside the part examined.
2. Use of a **bolus** near the part to be radiographed. This cuts the peripheral "black" which spreads over the anatomy, but also casts artifacts.
3. Use of a **compensating filter**:
 - a) a "black" area will be attenuated, such as the periphery of the shoulder with the "BOOMERANG";
 - b) a "white" area may be better penetrated while protecting the rest of the film, v.g. C-7 with the "INGOT".
4. Local **digital** adjustments on areas too bright or too dark, but not simultaneously with normal areas.
5. An **electronic threshold** cutting off overexposed areas of the image.
6. A **combination** of many factors.

MATERIAL AND ABSORPTION

1. **Rice, flour, etc.** do not absorb sufficiently.
2. **Water** and **serum bags** would be good mainly as boluses, but not practical. Near the trunk they would be large and heavy, although adding an opaque substance improves them.

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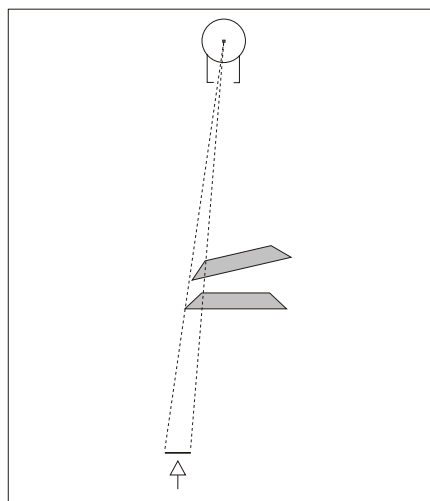
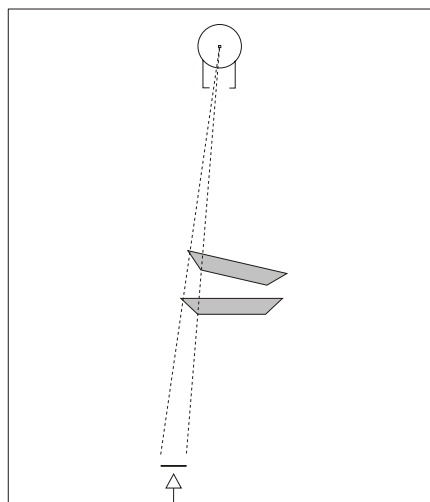
Gradient estompé.

Soft gradient.

3. Rigid **plastic** is good, but the tapered edge may break, thereby injuring a patient or a technician; it does not adhere vertically.

4. Metals, such as **lead** and leaded plastics, are generally too dense. **Copper** and **aluminum** are useful.

5. **OCTOSTOP® compensating filters** are made of **silicone rubber**. Their absorption is ideal at twice that of muscle, which permits placing them on only one side of the patient.

They are light to carry, easy to handle, readily identifiable and not cumbersome. They are positioned with precision close to the patient. If dropped, they do not break or harm anyone. In the vertical position, they adhere with very little pressure (wash with water if they slip). If too sticky, interpose tissue paper.

They resist alcohol and heat. They are waterproof.

6. PHARAOH boluses in polyurethane surround the head and the neck completely: they absorb like water.

GRADIENT OF ABSORPTION

It is proportional to the slope of the wedge: the steeper the gradient, the sharper and more obvious is the absorption. The more gradual it is, the less perceptible is its shadow. It is measured in degrees, or as a fraction:

BOOMERANG: 30°-45° or 1/1.7-1/1

INGOT: 45° " 1/1

GENTLE SLOPE 8° " 1/7

PRISM: 37° and 52° " 1/1.3 and 1/1.8

In addition, the gradient varies according to the orientation of the filter. It increases as the axis of the wedge becomes more parallel to the direction of the X-ray beam; it decreases in the reverse situation (see figures).

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
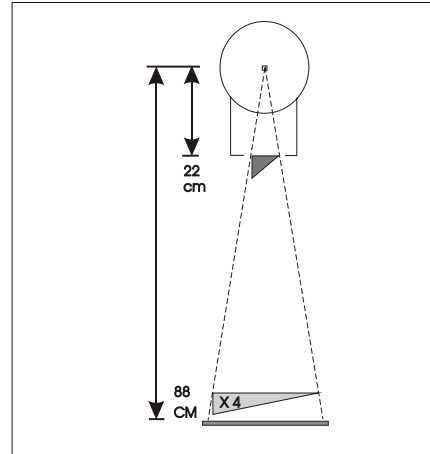
Gradient at collimator vs on patient

Bord effilé.

Tapered edge.

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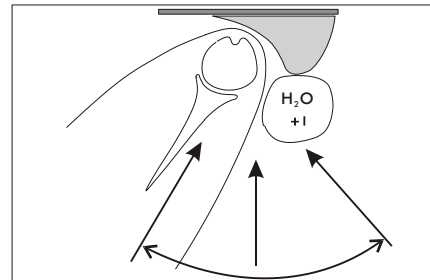Special tomography.

POSITION

1. *Behind the patient.*
2. *In front of the patient: saves radiation to the patient.*
3. *At the collimator, with Velcro[®] as an inexpensive means of attachment. Here the gradient diminishes by a factor of 4, since its width is magnified in the proportion of 88:22 cm (see diagram).*

SHAPE

1. *Curved at 90° for the BOOMERANG.*
2. *Rectilinear for other filters.*
3. *Adapted.*

TAPERED EDGE

It is fragile and must not be stretched or torn off with fingernails. It is nevertheless long lasting with reasonable care. Do not fold over, as this may create an artifact.

SPECIAL TOMOGRAPHY (axial of the shoulder)

When a superficial region is **very thick**, controlling its density is difficult: because the wide sweep of very "black" air coming from levels far from the cut, largely overlaps the anatomy, which then becomes much too dense. Nevertheless, two solutions are available:

- A. **Place a filter against the patient and add a bolus containing iodine.**
- B. **And/or use a highly absorbing filter at the collimator.**

STERILIZATION and DISINFECTION

Autoclave is recommended for sterilization; alcohol or Zephiran[®] for disinfection.

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**A compensating filter
to even out the density
of shoulder
radiographs**

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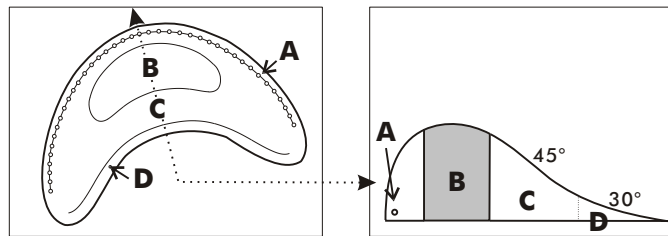
DESCRIPTION

A. **Metal chain** along the periphery indicates the presence and position of the filter on the film. It projects outside the anatomy, and reduces risks of legal litigations.

B. Area of **maximum** absorption: equivalent to +/- 9 cm of muscle. For large shoulders, cephalometry, etc.

C. Area of **intermediate** absorption: +/- 45° gradient, for smaller shoulders.

D. Area of **minimum** absorption: 30° gradient, v.g. for patella.

The shadow of the shoulder from the light localizer should fall at the desired absorption level B, C or D. The filter will stay in the vertical position with a slight pressure from the rotated trunk.

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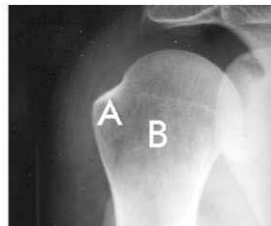
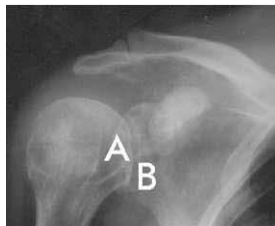
ROTATION OF THE HUMERUS

A. Greater Tuberosity
B. Lesser Tuberosity

The position of the tuberosities is controlled by the position of the elbow.

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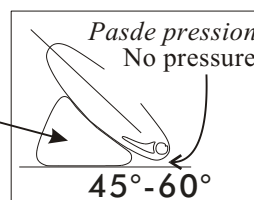
Two views at 90° are preferred for trauma cases.*

ROTATION OF THE TRUNK

The shape of the glena is controlled by the position of the trunk.

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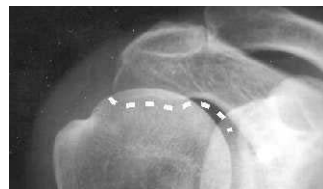
Lower border of the spine of the scapula....

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...merges with inferior border of the acromion.

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Decubitus requires more angulation than the upright.

Always position the filter according to the localizing light.

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KYPHOTIC PATIENTS

Upright requires less angulation than normally. Watch the shadow of the shoulder on the filter. Then, interpose a foam cushion.

Decubitus requires more caudad angulation than the upright.

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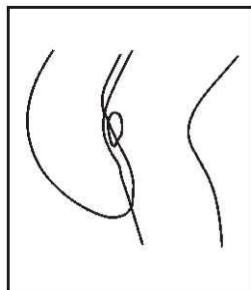
PATELLA and TIBIAL TUBEROSITY:

Use only a few cm (area D) of one of the tips. Do not cover the supra-patellar area to avoid a fluid artifact.

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CEPHALOMETRY:

Place maximum thickness (zone A) against nasion and chin. Use a high KV (90-100), a low MaS and slightly underexpose.

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D1-D3: when this portion is thinner than the rest of the thorax.

IMPORTANT

Only the wedge portion is normally superimposed, while the periphery remains outside. Since the tips are at 90° to each other, they might occasionally be superimposed, and create an artifact. This would be recognizable by the presence of the chain. If there is a risk of a misinterpretation, make a note on the film, or take a corrected exposure.

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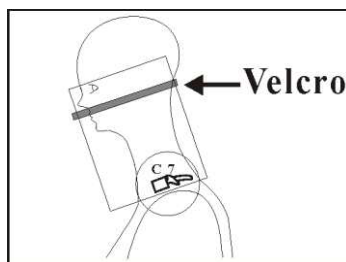
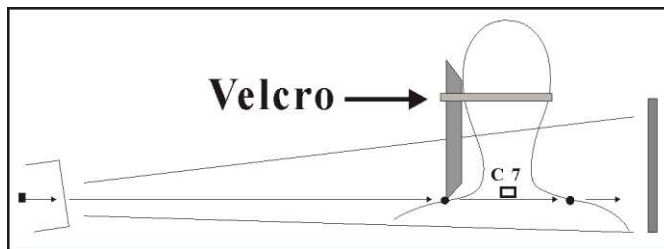
FORC-7

1. **The focal spot of the tube must be centered low, level with both shoulders (according to the "two-point" technique, cf. THE OCTOSTOP SYSTEM II, p. 7).**

2. Then, **without moving the tube**, aim the collimator higher to include C-1.

3. **Under direct medical supervision**, it is often necessary to **pull the arms down**. For a useful method, see page 10. **In the emergency room, look for fractures of the trunk or the extremities.**

4. In the vertical position, a Velcro® band holds the INGOT very well.

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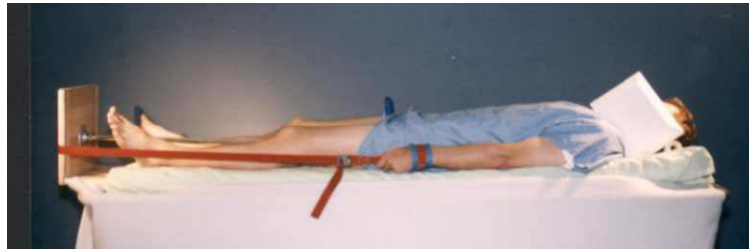
CHARACTERISTICS

1. Use **appropriate technical factors for the unfiltered region**, while keeping the **MaS high**, and the **KV low**, in order to diminish secondary radiation.

2. **Adapt** to local preferences. Consult your radiologist.

3. **Straight edges** would be easily identifiable and thus do not require a metallic tracer.

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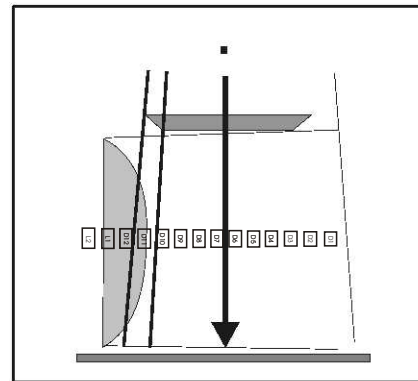
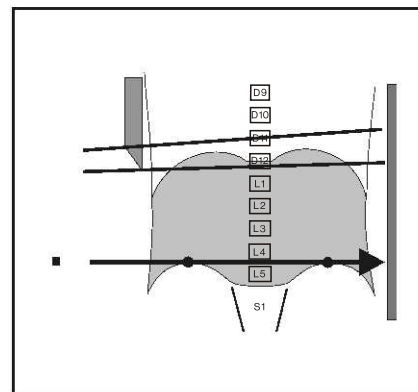
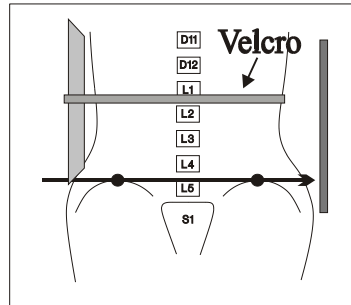
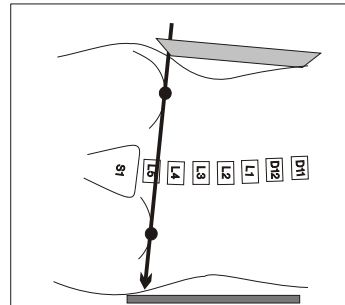
Decubitus

**LUMBAR SPINE,
to penetrate L-5**

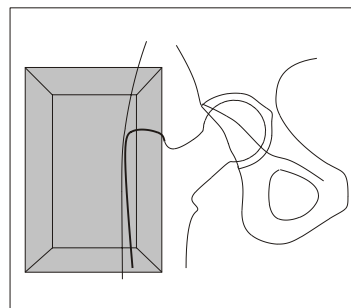
Only for women with a thin waist and
a wide pelvis.

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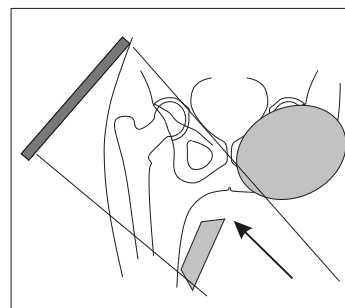
Lateral

HIP

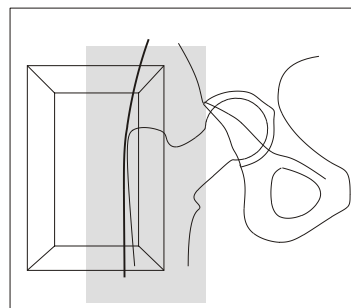
A.P. view, for the greater trochanter if
tissues are emaciated.

Profile view, to decrease the density of
the proximal 1/3 of the femur (stem of
prosthesis).

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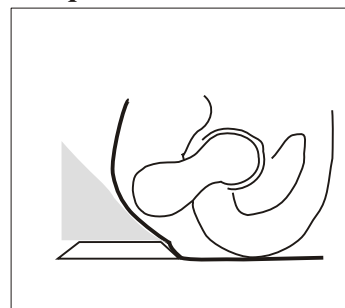
Cross section

HIP, skin folds

A triangular foam cushion squeezed
between the INGOT and the skin
erases the skin folds.

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HANDS AND WRISTS

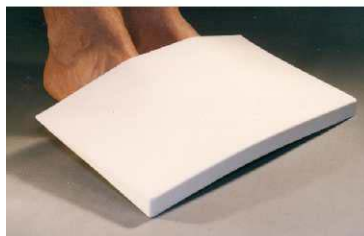
- 1. On the A.P. view, filter only the fingers, since there is little difference in thickness from the hand.**
- 2. For all other views, superimpose up to the metacarpals.**

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FEET

Place the toes near the thick end of the filter, for all views.

Interposed before the rays, a filter reduces radiation to the patient.

Pressure (with a small wood board) often straightens out the toes.

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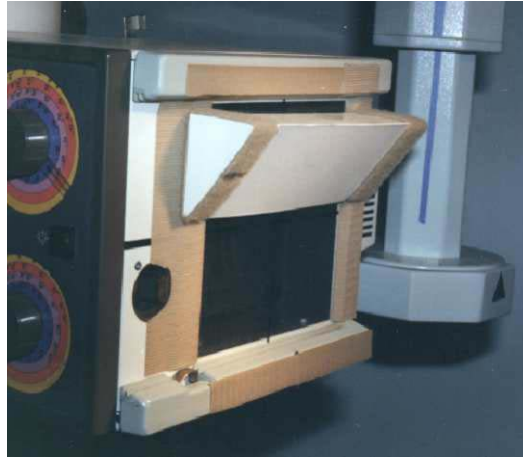
CHARACTERISTICS

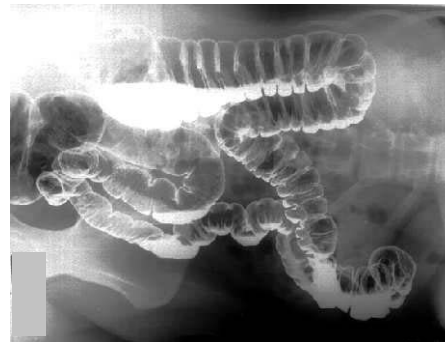
1. Has two gradients: one of 37°, for regular filtration; the other of 52°, for steeper filtration. When placed on the collimator, the gradient is 4 times less on the film (see p. 4). Velcro® is fixed to both extremities.

2. Prepare the collimator by placing the self-adhesive Velcro® around the window. If a track is present, use one or two supplied plastic spacer(s) nearby to raise the Velcro® at the level of the track.

3. The little prism may then be fixed in any position.

4. It may also be placed next to the patient.

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**ABDOMEN,
HORIZONTAL BEAM**
For sagging abdomen only. Filter
no more than 5 to 12 cm on the patient.

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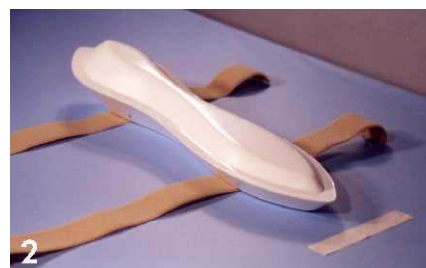
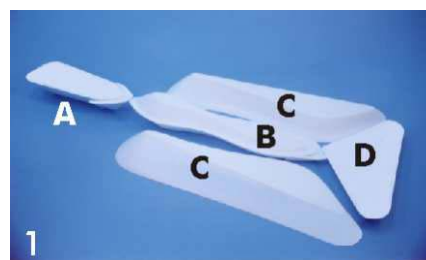
The RIDGES PLUS insure constant optimum density of digital arteriography (and venography) of the lower extremities down to the feet. By removing halation, they facilitate digital subtraction. With the cushions, they are an excellent immobilizer. See the video included with each kit (also available for free).

DESCRIPTION (Figure 1)

They are all thinner distally. Two incorporated midline opaque rulers allow measurements of site, length and caliber of lesions. The **inter-thighs filter (A)** has a pyramidal shape. Protect it with thin wrapping. The two **inter-legs filters (B)** are bilobed; the main one is thicker. The two **external ridges (C)** contour the calves and the dorsum of the feet; they also suit the upper extremity. A **triangular (D)** filter borders the sole of the feet. Extensions may cap midline filters for longer legs.

PREPARATION (Figure 2)

Place the two Velcro® bands on the table, felt facing upward. Lay the main inter-legs upside down, then the accessory one over, with all arrows in the direction of the arterial flow. Place a tape with the adhesive side upward at the level of the heels.

INSTALLATION (Figure 3)

The patient lays with his legs on the bands, while the inter-legs filters are re-positioned over them, the depressions against the calves. Stick the heels together with the adhesive tape underneath. Then tighten the straps around both filters and legs. If the knees are difficult to bring together, flex them slightly. Lean the lateral ridges against the legs and the feet. Push the triangle filter upside down against the sole of the feet. The inter-thighs filter will be placed before or after the puncture, with the top downward and the pointed end toward the knees.

IMMOBILIZATION (Figure 4)

Adhesive tapes are anchored to the sides of the table. Protect the cushions with a thin plastic sheet. A loop of adhesive tape draws each foot in external rotation against a prism-shaped cushion. Crossed adhesive tapes press down the pyramidal filter between the feet, preventing dorsal flexion and rotation. Finally, a transverse adhesive presses the upper part of the inter-legs filters between the knees that are squeezed between the lateral filters.

VERIFICATION

Under fluoroscopy or with films, make sure that all is centered and symmetrical; that knees and heels are close together; and that there is no halation.

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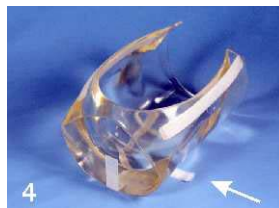
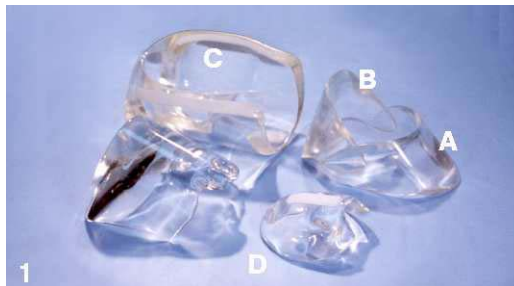
the PHARAOH

It is both a **compensating filter** and a strong **head immobilizer**. Made of flexible polyurethane, it has the same absorption as water, and is designed for digital arteriography, of the aortic arch, the carotids and the cerebral arteries, with and without subtraction. **See the video included with each kit (also available for free).**

The PHARAOH consists of four crystal clear elements (**Figure 1**): **A**) an **anterior collar**, marked with an "A"; **B**) a **posterior collar**, marked with a "P"; **C**) a **bonnet** and **D**) an **eye mask** with openings for the eyes. Also of two joining disks and Velcro®.

NECK COMPONENTS (collar)

With the patient sitting, the *posterior* part "P" (**Figure 2**), leans against the shoulders. The *anterior* filter "A", presses up against the jaw (**Figure 3**), overlapping the posterior one evenly on each side. The two filters are fixed together with adhesive tape on each side.

BONNET

An auto-adhesive transverse Velcro® stuck to the table (**arrow, figure 4**) anchors the bonnet exactly where the head will be examined. Then, the patient places his head into the bonnet (**Figure 5**), pressing against the top. The sides are tightened closer together with adhesive tape.

VIEWS

Many AP and OBLIQUE views are done only with the collar and the bonnet. For a profile view, the mask is laid delicately over the face.

IMMOBILIZATION

If an immobilization is required, the eye mask is joined to the bonnet through disks and short Velcro® strips (**Figure 6**). Long adhesives fix the top of the mask to the sides of the table (**Figure 7**).

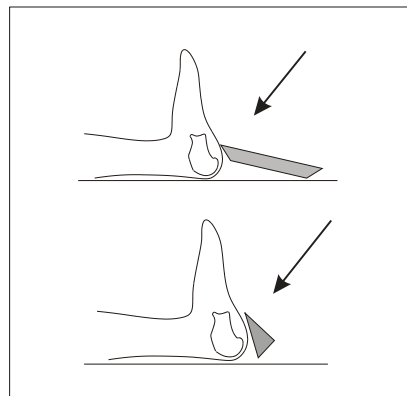
VERIFICATION

Fluoroscopy and/or films prior to the injection are important for high quality imaging, since a correction before the injection now will insure better results after.

ROTATION


Pivoting shields fixed to the collimator are a useful accessory to automatically adjust the width of the field between the AP and the LATERAL.

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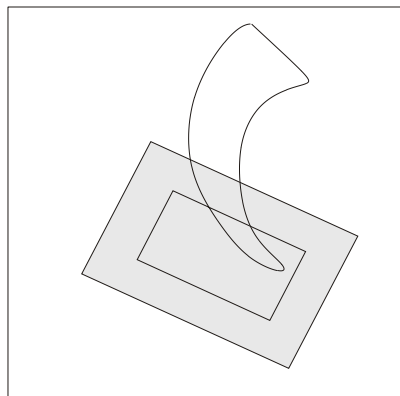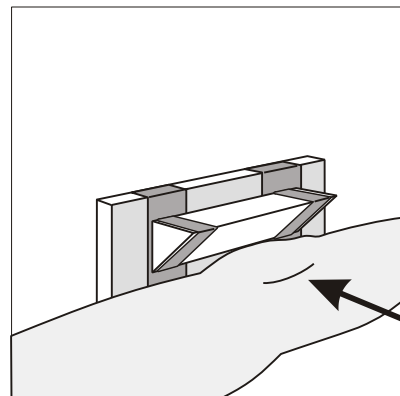
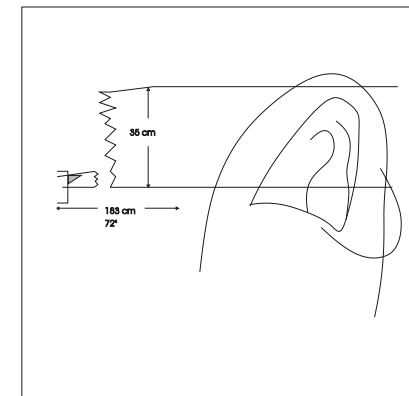
** Consultez votre radiologue.*

OCTOSTOP® filters are new. Do not hesitate to experiment. Here
are our suggestions. Send us yours.

The BOOMERANG may reduce overpenetration around the
occiput, and other parts of the skull. On the other hand, a double suction
cup helps holding it vertically against the table..

The LITTLE PRISM on the collimator may compensate the upper
chest of corpulent women having a pyramidal thorax, with high and large
breasts.

Again on the collimator, it may attenuate the lower 1/3 of the
humerus, or the femur.

Or near an orbit or the face, for certain oblique views, it
compensates lateral hyperpenetration .

The GENTLE SLOPE may be used in contact with the lower 1/3
of the humerus or of the femur, and sometimes over the thorax.

It may be cut in half with a sharp blade preferably in a wedge
fashion. Superimposing both parts doubles the gradient and the
absorption. They would be reunited for a bilateral examination.

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IMPORTANT PRECAUTIONS

1. Read this educational brochure attentively.
2. Do not filter when there is little or nothing to be gained.
At the beginning, conduct your own experiments.
3. In general, adjust technical factors for the unfiltered area.
4. If an artefact is observed, indicate it on the film;
or better, take another corrected exposure.
5. Observe local practices. Consult your radiologist.
6. Send us your suggestions.

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IMPORTANT

Must be used only by qualified personnel and under the direct supervision and responsibility of a physician. When applying traction, caution should be exercised when the following conditions are present or possible: fractures of the pelvis; spine or upper extremities; uro-genital or abdominal trauma; or all other circumstances that might be contraindicated.

OCTOSTOP[®] Inc. and its personnel do not accept any liability for how its products are used, for the indications, consequences or any situations directly or indirectly related to its products.

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