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**gentle  
pediatric  
immobilization  
since 1964**

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

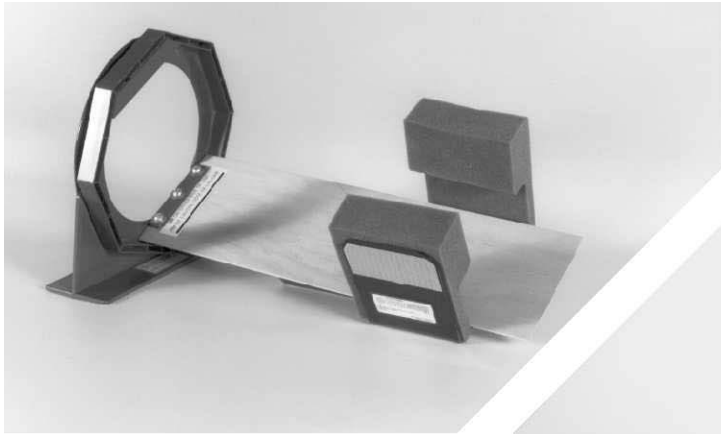
All rights reserved  
FOR ALL COUNTRIES  
®Registered trademark

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**A modular concept, extremely adaptable. Those who have tried it find it to be indispensable. At last, no more fights with babies!**

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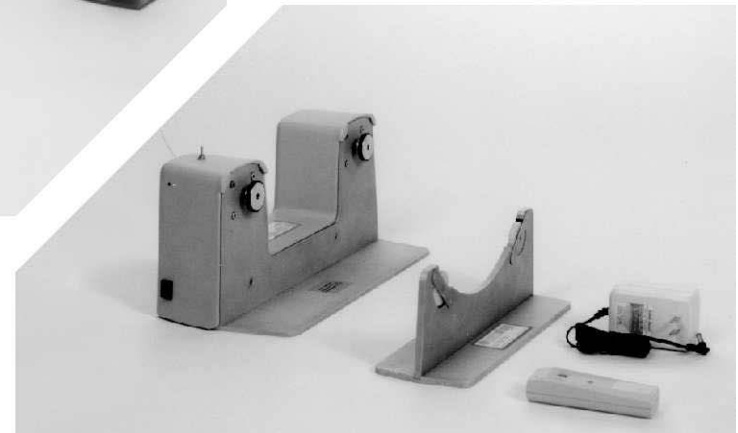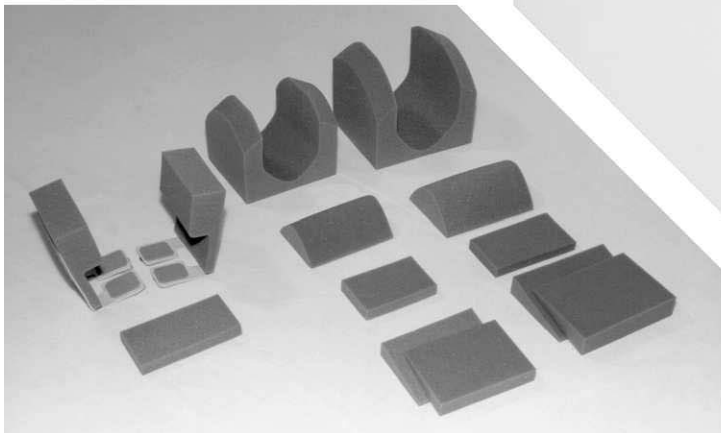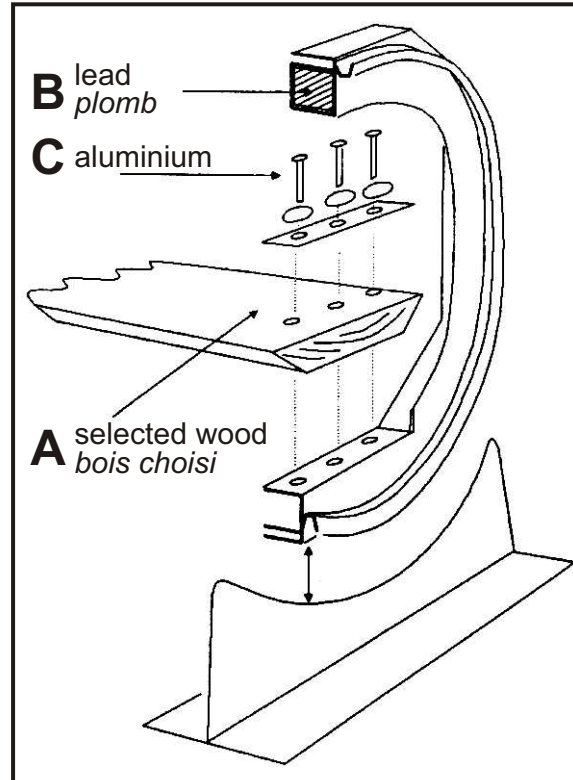
Figure 1

### THE BOARD

It is made of carefully selected wood (**Figure 1-A**), which is much superior to plastic in terms of radiolucency (less than 0.8 mm Al. equiv.), as it is virtually invisible on the anatomy. It can be disinfected with alcohol or Zephiran<sup>®</sup> solution. **Avoid sharp objects.**

Its width (6.5 inches) is ideal as it permits immobilization of the head as well as the trunk, and from new-borns up to 5 year-old children, regardless of size. Length: 40 inches. Width: 6.5 inches. A wider 10-inch option is available at 0.9 mm Al. equivalent.

Velcro<sup>®</sup> hooks glued underneath temporarily fix all restraints, which are thus detachable for cleansing. These hooks are self-adhesive and easily replaceable if stained.

### THE OCTAGONS

They are made of strong blue plastic, 11 inches in diameter. They provide a stable position every 45° around a complete circumference. A lead counterweight (**Figure 1-B**) balances the unit. They are open to accommodate the feet of tall children when necessary. An adjoining groove graduated every 15° adapts to the supports. Screws and locknuts are made of aluminum.

### MAGNETIC RESONANCE

No ferro-magnetic material is used (**Figure 1-B and C**). Plastic centering devices are fixed under the board.

IN MAGNETIC RESONANCE  
USE BETWEEN LINES

### THE SUPPORTS

The octagons slide easily on the supports, while their shape stops any spontaneous rotation. They permit positioning at any angle about an **iso-centered** longitudinal axis, without any sideshift. The object/receptor distance remains at a minimum. Subdivisions are indicated every 5°.

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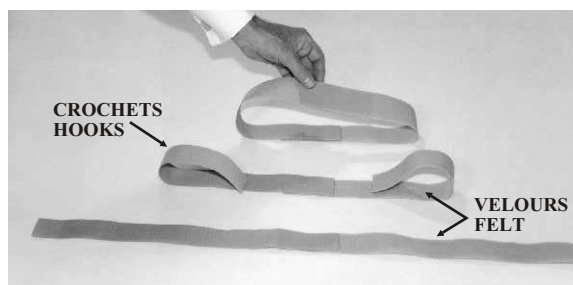
Figure 3

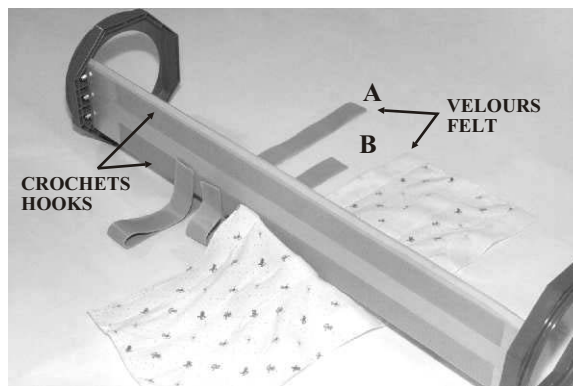
Figure 4

They adhere temporarily on self-adhesive Velcro® hooks under the board (Figure 4). They are all removable, washable, disposable, and replaceable at low cost.

## FOREHEAD STRAP (Figures 2-A, 4-A)

It is made of Velcro® felt, 36 inches long, and holds the head with its cushions. Both ends adhere to the hooks under the board. Do not overtighten if there is a possibility of a skull fracture.

## ARM STRAP (Figures 2-B, 4-B)

It is usually installed, but tightened only on very agitated children, as "L" cushions usually suffice. Made of Velcro® felt, 20 inches long, it is passed behind the neck, never in front, and over the arms. Both ends adhere under the board.

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They are made of flannel reinforced with sewn Velcro® straps. Knees and elbows cannot escape between the straps.

When open (Figure 4), the "felt" portion faces inwards on the left side of the baby; but the "hook" one, on the right of the baby, faces outwards, to avoid scratching the skin. They close and open near the technician.

## OCTOPAQUE® STRAPS (Figures 2-E, 3)

Like the blankets, the felt part (on the left side of the baby) faces towards the baby, while the hook one (nearby, on the right of the baby) faces outwards, or away from the baby to avoid scratching the skin. Press a few inches firmly together (Figure 3), and check adhesion with a good longitudinal pull on both parts. Total length: 31.5 inches.

## CLINGING BANDAGES (Figure 2-F)

Some prefer to use this material around the baby and the board. It gives excellent results.

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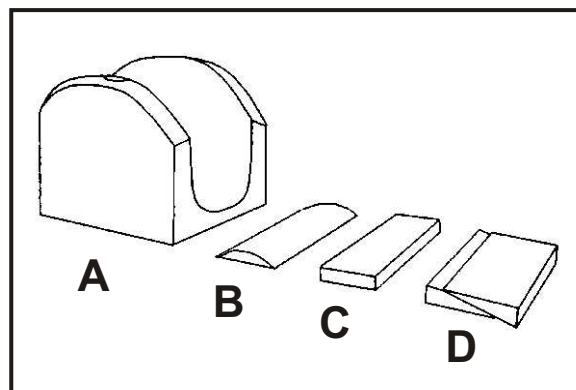
Figure 5

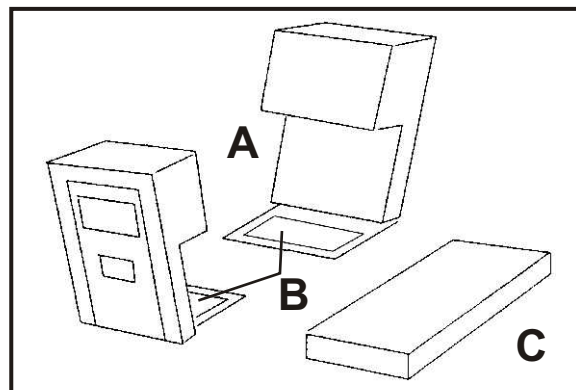
Figure 6

Soft, comfortable and of chocolate colour, they are always interposed between the restraints and the skin to protect it.

## "U" CUSHIONS (head only) (Figure 5)

The special form of cushion (A) displays the face to allow observation for the best time for exposure. In addition the surface landmarks are visible to apply the "two-point technique" (see page 7). The base is thin to keep the head close to the film. The tension of the forehead strap or of the bandages is distributed all around so that it need not be excessive. There is always a return to the original position after a slight movement.

**LARGE:** for 6 months to over 8 years.

**SMALL:** from birth to 12 months.

The half-cylinder (B) when placed at the bottom of the "U" flexes the head. Under the neck, with other cushions, it produces extension.

The forehead rectangle (C) is larger than the strap to afford excellent protection.

The triangles (D) help to correct any obliquity or inclination when needed.

## "L" CUSHIONS (for head and arms) (Figure 6)

They are notched (A) to hold the arms alongside the head. Thus, tightening the arm strap is not always necessary (page 3). They are fixed under the board with their felt (B).

A narrow rectangle (C) protects the forehead.

### IMPORTANT:

- 1- Avoid staining the cushion, even if they are washable with water, Zephiran® or alcohol.
- 2- Protect them against heat and acetone.
- 3- Handle the plastic gently, and remove it carefully under the board.

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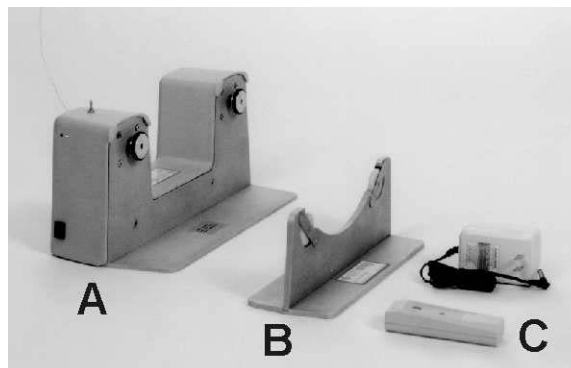
Figure 7A

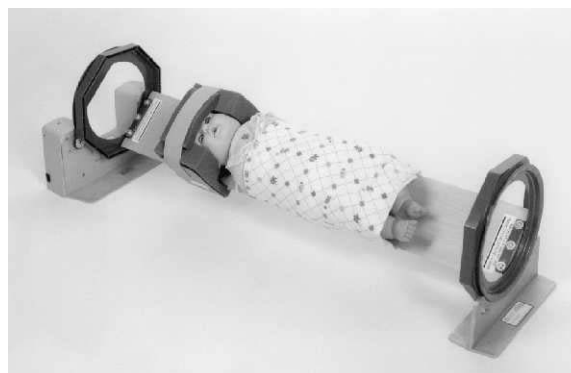
Figure 7B

A dependable unit, simple, light and portable. It operates on any X-ray table without any adaptation.

## MODULES (Figure 7-A)

The motor module (A) at the head end (left) incorporates two wheels, each driven by a motor operating on a rechargeable battery. In an emergency, the main switch stops all motion. Another local switch may control rotation, and overrides the remote control. A second support (B) has two free-spinning wheels.

The UNIVERSAL OCTOPAQUE® is simply placed on these four wheels (Figure 7B). It functions immediately at 5 rpm.

## WIRELESS REMOTE CONTROL

The command module (C) is a small radio-transmitter. It operates on a 9V alkaline battery. Pressing one of the two buttons emits a radio signal received by a receptor in the motor module, which in turn controls rotation in either direction. Starts and stops are immediate. The desired angle is set and maintained with precision, by finger-tip control, from anywhere in the examination or control rooms.

## AUTONOMY

A blinking red light warns before the battery is too low. A 110 V.A.C. wall plug-in charger (D) then fills the battery, even while working. The charger usually remains connected during nights and weekends, or permanently. Battery power is sufficient for an average day's work.

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**THE OCTOROLL® II MUST NOT BE USED IN  
A MAGNETIC RESONANCE ENVIRONMENT**

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**IMPORTANT: PLACE THE TOP OF THE HEAD 10 INCHES FROM THE OCTAGON TO AVOID SUPERIMPOSITION ON IT.**

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Fine-grid cassettes (more than 75 lines per inch) are much preferable to "Bucky" films: they permit very fast exposures, minimize the object-film distance and give excellent detail without showing the lines, providing that the focus-distance and the 90° incidence are respected.

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## **PROFILE (Figure 8)**

Use a horizontal beam, and a cassette on a vertical support.

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## **REGULAR A.P. (Figure 10)**

The petrous bones are usually projected in the orbits. Modify according to local preferences.

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## **TOWNE'S VIEW (Figure 12)**

Place the half-cylinder cushion under the head to flex it. Angle the tube approximately 30° toward the feet.

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## **INVERTED WATERS' VIEW (A.P.) (Figure 14)**

Angle 30° to 40° toward the head. Raising the neck with a cushion helps reduce angulation.

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
## **REGULAR WATERS' VIEW (P.A.) (Figure 16)**


For ambitious technicians!!! Orbits are smaller, sinuses have better details: usually easy if one is patient enough to let the baby calm down, and if good immobilization is achieved. Usually 30° to 40° caudad.

**Figure 8**

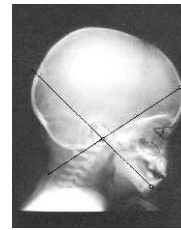
**Figure 9**

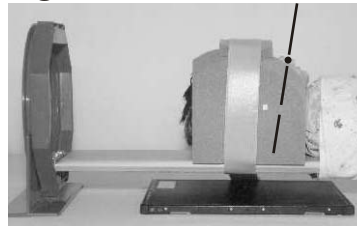
**Figure 10**


**Figure 11**

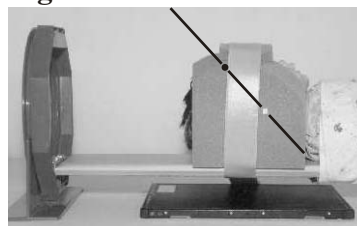**Figure 12**

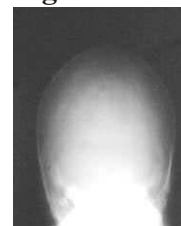
**Figure 13**

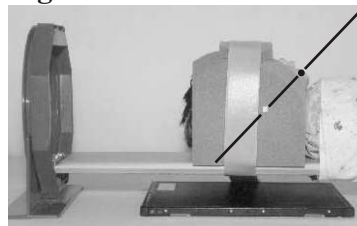**Figure 14**

**Figure 15**

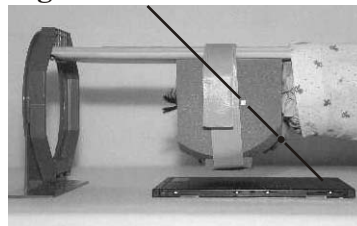**Figure 16**

**Figure 17**

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# "TWO-POINT" TECHNIQUE FOR SKULL VIEWS

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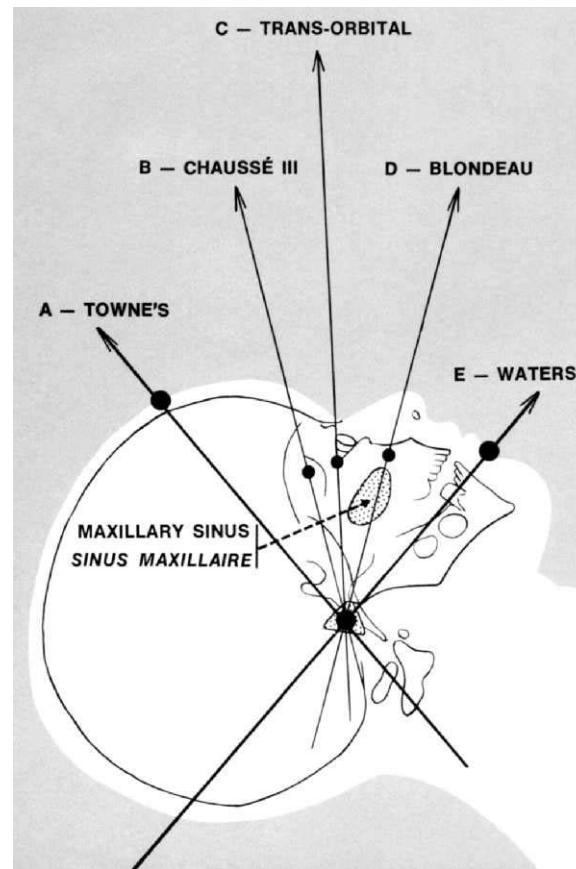
Figure 18

## 1. FIRST "COMMON POINT"

Mark the position of the external auditory meatus on the cushion with a small piece of adhesive, or on the bandages with a ball-point or felt pen.

## 2. SECOND POINT

- |                               |                      |
|-------------------------------|----------------------|
| A. The hairline:              | TOWNE'S              |
| B. The supra-zygomatic fossa: | CHAUSSE III          |
| C. The outer canthus:         | TRANS-ORBITAL        |
| D. The cheek:                 | BLONDEAU (for orbit) |
| E. The mid chin:              | WATERS'              |

## 3. ALIGNMENT OF TWO POINTS WITH THE FOCAL SPOT

Trace an imaginary line with a pointer, a pencil or a flash-light through these two points, and extend it straight to the focal spot of the tube. Then turn the collimator towards the desired area. Finally, make the beam as narrow as possible.

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# IMMOBILIZATION FOR TRUNK EXAMINATION

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Figure 24

## PREPARATION

To the back of the upright board (**Figure 19**), fix the "L" cushions (**A**), the forehead strap (**B**) and the blankets (**C**) with the long side to the left, felt towards the board, short side to the right, hooks towards you.

Place the unit in radiography position (**Figure 20**). Place the arm strap (**D**) near the cushions, felt facing downward. Then place the baby and immobilize the trunk and the lower limbs with a blanket (**Figure 21**).

## THE ARMS

If used, the arm strap must always be **behind** the neck (**Figure 21**). Then pass it successively over each arm and fix it tightly under each side of the board (**Figure 22**). **This controls the whole body.**

## THE HEAD

Place the arms in the notches of the "L" cushions. Apply the forehead cushion while pushing the hair back. Wrap the forehead strap tightly around all cushions and fix it under the board (**Figure 23**).

## OTHER METHOD, WITH BANDAGES (**Figure 24**)

Some prefer 5-8 inch wide bandages: raise the board on a block; then wrap them around the trunk, the arms, **behind the neck**, and around the head and arms, with or without the cushions.

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Figure 26

## INTRAVENOUS PYELOGRAPHY

The arms and the head can be free. Never leave a baby unattended. Use a grid cassette of 75 or more lines per inch for best details and contrast, and a fast exposure (near 1/120th of a second).

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## CHEST

Use a very fast exposure.

Expose in deep inspiration. Remember that a baby cries in expiration. Inspiration occurs immediately after, and is short. Practice exposure with an instant switch saying:

".....eee.....  
cry (expiration)

...he...  
small pause (inspiration)

...shoot!"  
exposure (end of inspiration).

In other words: "At the end of a long cry, small pause, and shoot!" the desired area.

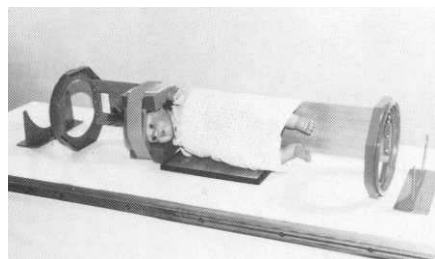
Figure 29


Figure 30

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**PREVENT SPREAD OF SPILLS.  
CONTAIN THEM IN WATERPROOF  
PADS, DIAPERS AND TISSUE  
PAPER.**

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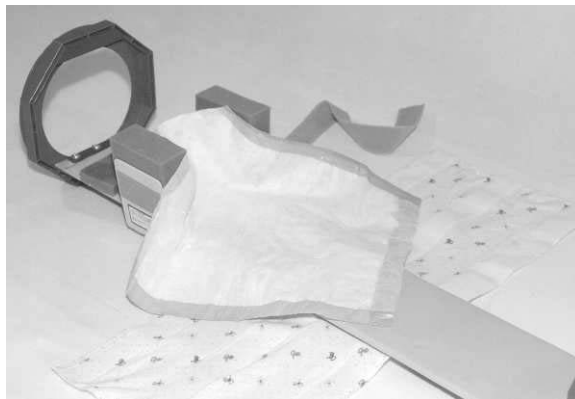**Figure 31**

**OESOPHAGUS, UPPER GI., and CARDIAC  
SERIES.**

Put a waterproof absorbing pad about the neck, and  
use many tissues near the mouth and the neck  
(Figures 31 and 33).

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**Figure 32**

**VOIDING CYSTOGRAM and  
BARIUM ENEMA**

Place waterproof absorbing pads, and thick  
absorbent diapers on the board before placing  
the baby.

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**Figure 34**


## **OESOPHAGUS, STOMACH and DUODENUM**

Prior to the examination, the head must be immobilized in slight rotation to the right, to prevent superimposing the bottle over the oesophagus. Use plenty of tissues around the mouth and cheeks to collect drips.

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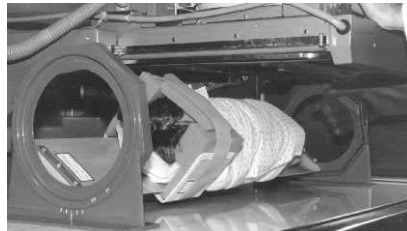
**Figure 36**

## **HANDLING**

With the right hand, turn the foot octagon on its supports: the baby will remain centred. The left hand operates the fluoroscope.

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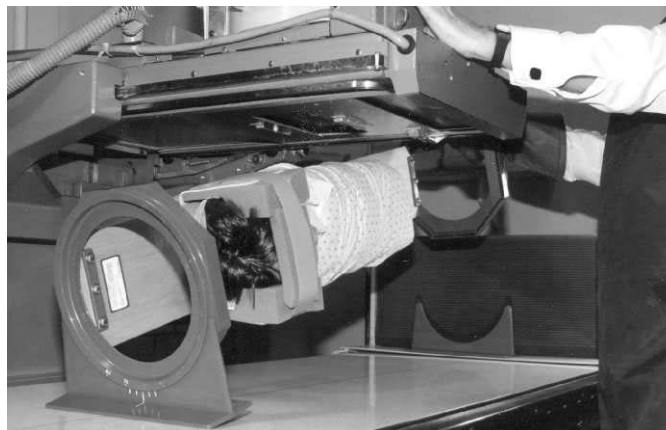**Figure 37**

**NOTE:** a hidden duodenal bulb is usually well shown by raising the feet some 15 inches (Figure 37).

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**Figure 40**

## **RETENTION OF BARIUM**

The barium is practically always retained **as long as the knees are held down in full extension.**

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**Figure 41**

## **CONTROLLED EVACUATION OF BARIUM**

It is preferable that evacuation takes place outside the board and the restraints to avoid staining them. Therefore one must continuously hold the knees down in full extension, and while the baby is detached and slipped off the board. Then remove the canula, and let the evacuation proceed in the diaper and the waterproof pad.

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## MINI-PAQUE UNIT

This small model of 20 inches long is for newborns and prematures. It adapts to most incubators and bassinets (Figure 42).

## OBSERVING THE COMFORT OF THE HEAD

Ask someone to press your hand firmly inside a head cushion (Figure 43).

## TIME SAVERS

Speed may be increased by preparing and releasing babies outside the examination room on a mobile table with a second unit, and by exchanging them between examinations (Figure 44). Never leave a baby unattended in so doing.

- MAGNETIC RESONANCE
  - COMPUTER TOMOGRAPHY
  - NUCLEAR MEDICINE
  - ULTRA SOUND
  - INTERVENTIONS
  - RADIOTHERAPY
  - EMERGENCY ROOMS
  - BLOOD TEST UNITS ...Etc.
- are all facilitated by the **OCTOPAQUE® UNIVERSEL** unit.

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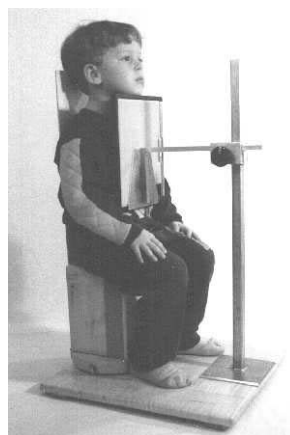
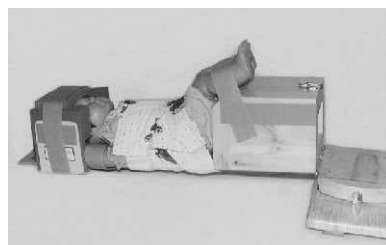
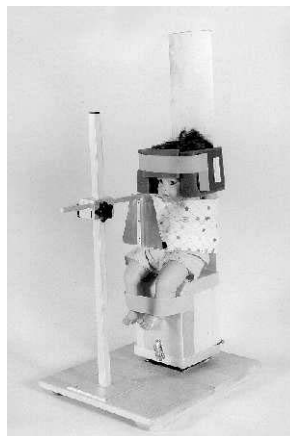
# the OCTOSTOP® CHAIR

Chest, cardiac series (and possibly sinus examinations) are performed with this simple chair, exactly as with the **UNIVERSAL OCTOPAQUE®**, since it incorporates a perfectly identical board.

## **IMPORTANT**

Fasten the latch securely in the vertical position. Continuously monitor patients. Never place them in a position that may compromise their safety.

## **SMALL BABIES**

It is easier to immobilize them horizontally. The radiographs are taken in this position, with a horizontal beam and a vertical cassette. Or they are placed upright.

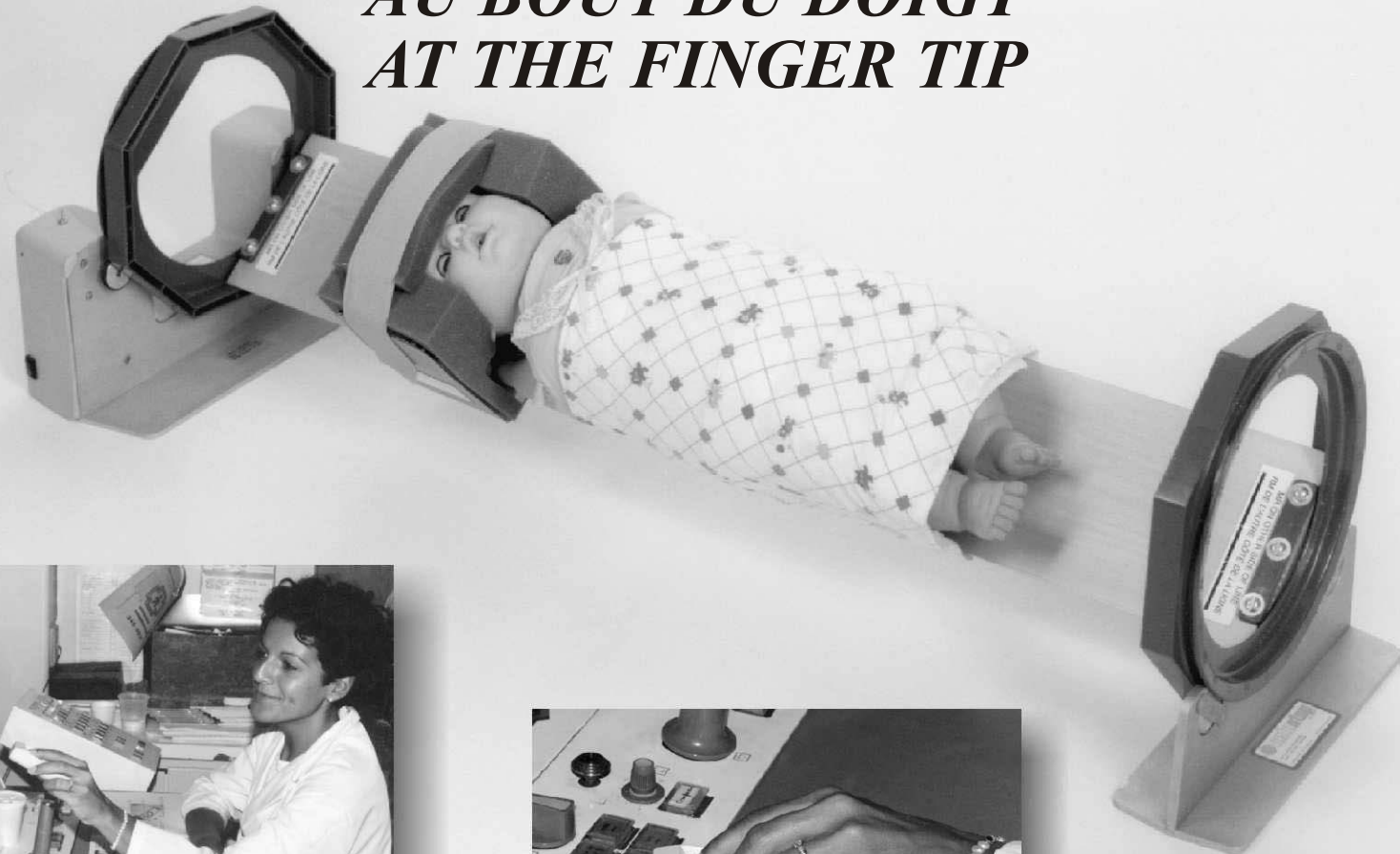
## **OLDER CHILDREN**

Immobilize them according to needs.

Localizing notches indicate every 45°.

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## GENERAL

1-The UNIVERSAL OCTOPAQUE®, OCTOROLL® II and MINI-PAQUE units and their techniques must be used under the responsibility of a radiologist or other physician. In case of doubt, paramedical personnel must seek advice. OCTOSTOP® INC. assumes no direct or indirect liability in the use of its products.

2-This brochure is only a guide for the better use of the OCTOSTOP® System. It must be adapted to each situation and to local usage.

## TECHNICAL

3-Never leave a baby **alone and unattended**, even when immobilized on an apparatus.

4-Avoid compressing the upper respiratory tract, by always passing straps and bandages **behind the neck, never in front**, or too close to the chin or shoulders.

5-The **upright position** is possible, but hold the board carefully manually or with the vertical support, and be sure that the baby does not slip down or become distressed.

6-The patient should not be placed in any position that could compromise its safety.

7-Attach all blankets and straps completely: they are radiolucent, but they may show as a slight artifact if bent or projected "on end".

8-Fine-lined grid cassettes are excellent: they permit a fast exposure (approximately 1/120th of a second); the lines are invisible providing the focal/grid distance and the perpendicularity are respected.

9-The plastic covering (optional) is delicate: avoid sharp objects; if torn, repair immediately with cyanoacrylate glue and the extra piece supplied; seal perfectly since opaque substances must not penetrate inside. The sides of the board are fragile: do not hit them. Bare wood may absorb some opaque substances. Repair it immediately if needed by sanding and varnishing.

10-Prevent contamination. Disinfect with alcohol or Zephiran® when needed.

11-Do not (over-) tighten a traumatized skull, to prevent complications.

## RADIOLOGICAL AND MEDICAL

12-The interpretation of an image and the result of a treatment or technique are the sole responsibility of the professionals in attendance.

13-A slight posterior angulation of C2 on C3 on the lateral cervical view, and a slight buckling of the trachea posteriorly and to the right are normal, since the head is immobilized with a slight flexion.

14-Normal pre-cervical soft tissues often bulge markedly.

15-Wood grain may show on oblique views; if an artifact is suspected, please take films with and without the board and its accessories. The plastic covering (optional) may be visible at 45°; in profile, it is visible outside the body.

Must be used only by qualified personnel and under the responsibility of a physician. OCTOSTOP® Inc. and its personnel do not assume any liability regarding the use, indications, consequences or any situation directly or indirectly related to its products.

**IN MAGNETIC RESONANCE USE ACROSS LINE FROM OCTAGON**

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COMPLIMENT OF

1875A, BOUL. INDUSTRIEL  
LAVAL H7S 1P5 CANADA  
TÉL. 450 668-2100 FAX. 450 668-1990  
WWW.OCTOSTOP.COM  
EMAIL: OCTOSTOP@OCTOSTOP.COM