

**BARD**

*Advancing the Delivery of Health Care.®*

## **Bard® Touchless® Plus Intermittent Catheter System**

**Leading the way to independence**



## Trust



Bard works closely with leading urologists to develop innovative products that enhance patient safety, convenience and ease of use. We are committed to using superior materials, then engineering them for the comfort and reliability needed to lead a life of independence.

Bard has been a leader in urology since 1907, and remains the product of choice for hospitals, rehabilitation facilities and consumers. With a broad range of products that set the standard for quality and innovation, Bard is dedicated to developing products that improve patient care and reduce costs associated with urinary tract infections.

Trust the choice of medical professionals.



### Ease of Use

Easy to insert, easy to use, easy to remove – the features needed to enhance convenience and usage.

- Catheter cap has pull ring for easy removal – especially for people with limited dexterity.
- Pre-lubricated catheter reduces discomfort during insertion.
- Large 1100cc collection bag is durable, virtually eliminating the risk of leaks.



### Safety

The Bard® Touchless® Plus Intermittent Catheter System is a closed system, designed to help reduce the risk of urinary tract infections.


- Insertion tip positions the catheter in the urethra, bypassing the meatus to help reduce UTIs.
- Patented catheter guide allows control of catheter insertion with the touch of a finger.
- Tapered neck design decreases excoriation associated with sharp corners.



**Sterile, Easy to Use:**

1. Remove cap 1 from insertion tip using pull ring.
2. Slide the pre-lubricated catheter 2 halfway into the insertion tip 3 and insert into urethra.
3. Holding the patented control guide 4 with non-dominant hand, use dominant hand to grasp catheter through bag 5 and push catheter into urethra.
4. Once bag is filled, withdraw catheter through insertion tip, snap on cap and dispose.

*Complete instructions for use included in product packaging.*

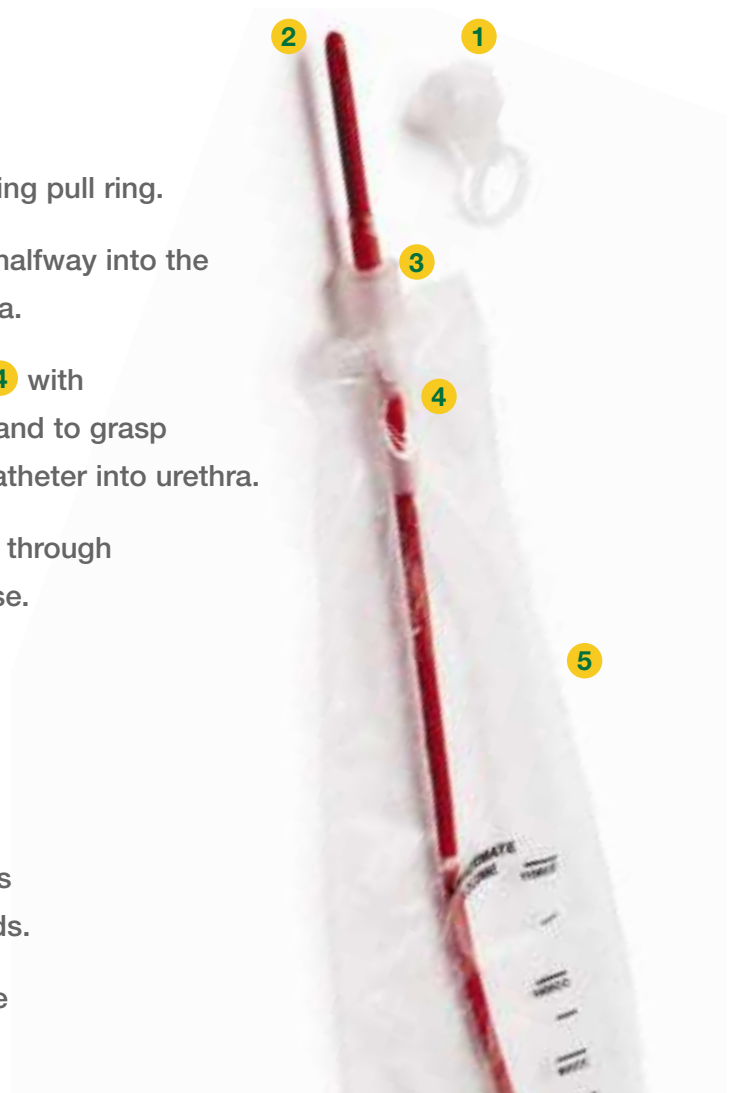

**Your System, Your Choice**

- Choice of red rubber or vinyl catheters based upon your preference and needs.
- Variety of catheter sizes, and available with or without accessories.

Product Number	Description	French Size	Latex-Free	HCPC Code
<b>Accessories included – 3 PVP swabs, 2 latex-free gloves, underpad</b>				
4A5108	Vinyl	8 Fr.	Yes	A4353
4A5110	Vinyl	10 Fr.	Yes	A4353
4A5142	Vinyl	12 Fr.	Yes	A4353
4A5144	Vinyl	14 Fr.	Yes	A4353
4A5042	Red Rubber	12 Fr.	No	A4353
4A5044	Red Rubber	14 Fr.	No	A4353
4A7044	Red Rubber Coude	14 Fr.	No	A4353
<b>Accessories not included (catheter and collection bag only)</b>				
4A6110	Vinyl	10 Fr.	Yes	A4353
4A6142	Vinyl	12 Fr.	Yes	A4353
4A6144	Vinyl	14 Fr.	Yes	A4353
4A6044	Red Rubber	14 Fr.	No	A4353

*Bard does not guarantee reimbursement. Refer to official code books for complete definition and absolute accuracy of HCPC Codes.*

**For more information, please contact us at 1-800-526-4455 or visit [www.bardmedical.com](http://www.bardmedical.com).**



**Bard Medical Division • C.R. Bard, Inc. • Covington, GA 30014**

Please consult product labels and inserts for any indications, contraindications, hazards, warnings, cautions and directions for use. Bard and Touchless are registered trademarks of C.R. Bard, Inc. or an affiliate. U.S. Patent No. 4,692,154 © 2006 C.R. Bard, Inc. All Rights Reserved. 0602-01 R 03/06 THP P03/06 5K Printed in U.S.A.