

# 3M™ Scotchcast™ Wet or Dry Cast Padding

*For use with 3M™ Synthetic Cast Stockinet and  
3M™ Scotchcast™ Plus Casting Tape*



*It's all you need for **WET AND DRY** casts.*

Patients can now enjoy the freedom to perform their normal daily activities including showering, bathing, swimming or using a whirlpool without worrying about getting their casts wet.

## **3M™ Scotchcast™ Wet or Dry Cast Padding:**

- Is completely immersible.
- Sheds water and feels comfortably dry to patient in approximately 1 to 3 hours.\*
- Is easy to apply and remove, and requires no additional training or tools.
- Is convenient in that it enables you to stock only one padding for all your patient needs.
- Is hypoallergenic.
- For use with 3M™ Synthetic Cast Stockinet and 3M™ Scotchcast™ Plus Casting Tape.

*\* See Patient Information pamphlet for patient instructions.*



**3M** Health Care

# 3M™ Scotchcast™ Wet or Dry Cast Padding

## No new application and removal techniques to learn

3M™ Scotchcast™ Wet or Dry Cast Padding is applied the same way and has the same performance characteristics as our current padding, so you won't have to learn new application techniques. Plus, the cast may be removed using the same methods currently used in the industry.

In addition:

- Product conforms well to the area being wrapped
- No special protection needed during removal

## Freedom and convenience

Your casted patients can now enjoy the freedom to perform daily activities. They have the opportunity to shower, bathe, enjoy swimming or a whirlpool without covering their casts.

**Note:** The blue yarn in the core of the product indicates that you are using 3M™ Scotchcast™ Wet or Dry Cast Padding.

To achieve best results use with 3M™ Scotchcast™ Plus Casting Tape and 3M™ Synthetic Cast Stockinet.

### Attention:

1. Patients with broken or compromised skin conditions should not get casts wet.
2. Patients must maintain their casts in suitable positions for drying.
3. Because it is difficult for infants to maintain proper positions for cast drainage, infants should not get their casts wet.
4. Because getting water to drain from casts on young children can be difficult, exercise caution when assessing whether young children should get their casts wet.
5. Patients who lack sensation in their extremities should not be allowed to get their casts wet.
6. Patients with fungal infections, such as athlete's foot (tinea pedis) should be instructed to use a waterproof cover over their cast while bathing.
7. Swimming, especially with a leg cast, may be difficult. For safety, deep water should be avoided.


## Ordering Information

### 3M™ Scotchcast™ Wet or Dry Cast Padding

Catalog No.	Size	Each/Bag	Bags/Case
WDP2	2 in. x 4 yd. 5 cm x 3.6 m	20	4
WDP3	3 in. x 4 yd. 7.5 cm x 3.6 m	20	4
WDP4	4 in. x 4 yd. 10.1 cm x 3.6 m	20	4
WDP6	6 in. x 4 yd. 15.2 cm x 3.6 m	20	4

### 3M™ Synthetic Cast Stockinet

Catalog No.	Size	Each/Case
MS01	1 in. x 25 yd. 2.5 cm x 22.8 m	1
MS02	2 in. x 25 yd. 5 cm x 22.8 m	1
MS03	3 in. x 25 yd. 7.5 cm x 22.8 m	1
MS04	4 in. x 25 yd. 10.1 cm x 22.8 m	1
MS06	6 in. x 25 yd. 15.2 cm x 22.8 m	1

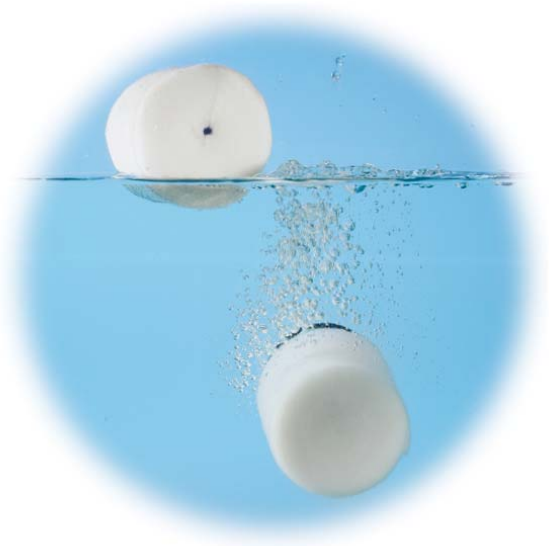

See Patient Information pamphlet for additional information.

To obtain a Patient Information pamphlet or for more information, contact your 3M representative or call: 3M Health Care Customer Helpline at 1-800-364-3577 or your local 3M Distributor



### Health Care 3M Canada

Post Office Box 5757  
London, Ontario N6A 4T1  
<http://www.3M.com/HealthCare/>

### Health Care

3M Center, Building 275-4W-02  
St. Paul, MN 55144-1000  
U.S.A.  
1 800 228-3957

1 800 563-2921 Ontario  
1 800 361-4488 Quebec  
1 800 361-4488 Maritimes  
1 800 661-8126 Western Canada

3M and Scotchcast are trademarks of 3M.  
Used under license in Canada.  
Printed in Canada © 3M 2006. All rights reserved.  
0511 CP23915E