

EasyTrack FS | v3

AVAILABLE SLINGS



HAMMOCK SLING
For most transfers.
Head-rest incorporated.
Suitable for most individuals.
Excellent for bathing.



QUICK-FIT SLING
For general transfers.
Perfectly suited for stocky
or obese patients with
strong hips or large thighs.



HYGIENIC SLING
Ideal for transferring to the
toilet. It also permits an easy
way to change, dress or
undress the individual.

	SMALL	MEDIUM	LARGE
Suggested weight	45-100 lbs (20-45 kg)	100-200 lbs (45-90 kg)	200-600 lbs (90-272 kg)
Suggested height	4'-5'11" (120-180 cm)	5'-5'11" (152-180 cm)	6' + (181 cm +)
Hammock	THA-S	THA-M	THA-L
Hammock 6	THA6-S	THA6-M	THA6-L
Quick-fit	TIR-S	TIR-M	TIR-L
Hygienic	THY-S	THY-M	THY-L



EasyTrack FS system # 9720010 Includes :
Extendable rail with trolley, 2 telescoping posts,
2 foot plates.

The system has **5** easy-to-assemble components.

Optional protective carrying bags makes the system easy to transport.



Bag Kit # 002.17010 Includes :
2 Post Bags: # 002.17000
1 Base Bag: # 002.17100
1 Rail Bag: # 002.17250
Bags also available individually



BHM
Because Health Matters

Canada
2001 Tanguay, Magog
Québec J1X 5Y5
5155 Spectrum Way, Suite #33
Mississauga, Ontario L4W 5A1
Europe
PO Box 28993, London SW14 8YL
UNITED KINGDOM
☎ 819 868-0441
☎ 819 868-2249
bhm@bhm-medical.com
www.bhm-medical.com

BHM Medical Inc. Reserves the right to change or discontinue any specifications, design, features, model or accessories shown without notice.

009.17000 Printed in Canada

EasyTrack FS

Featuring the V3 portable lift

A true portable lift and transfer system for the home that's easy to transport and quick to assemble.



BHM
Because Health Matters

EasyTrack FS

The Easytrack FS is a free-standing, versatile solution for all types of point-to-point transfers - wheelchair to bed, bed to chair, even in the bathroom, anywhere there is a lift and transfer need.

The Easytrack FS has a safe working load of 440 pounds, or 200 kilograms. That's exceptional for a portable product making it suitable for use in most any home.

This equipment was designed to allow those with a mobility impairment to continue to live at home and enjoy many of your normal activities.



1

The light weight aluminium bases feature adjustable levellers to ensure that the unit remains perfectly stable and secure when assembled.




2

The posts have been designed to be self-aligning. Components only fit together one way to prevent any assembly errors and ensure the safety of the system.



3

The rail can be extended up to a width of 115 inches (3 meters). That is more than enough to span a king size bed.

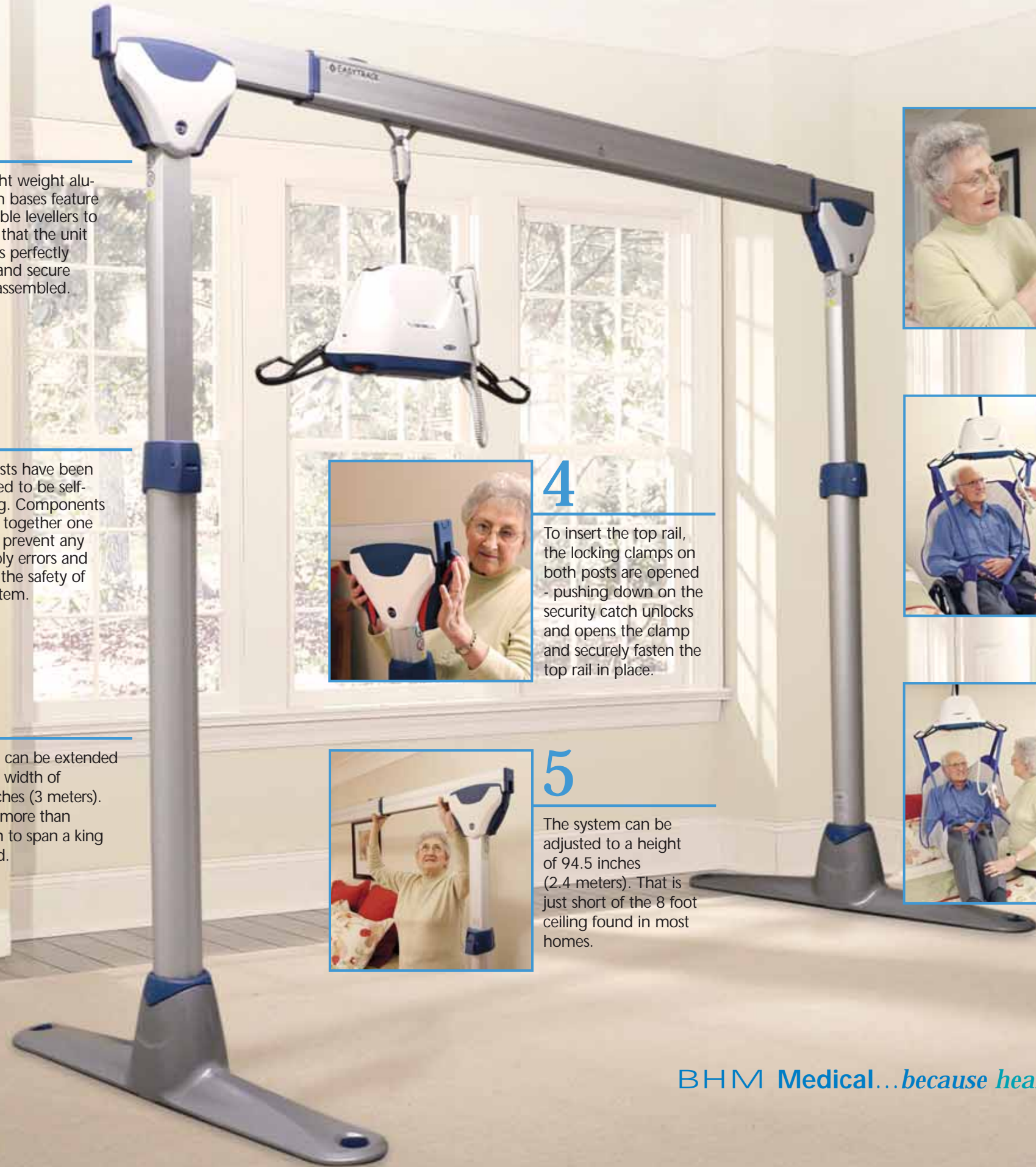

4

To insert the top rail, the locking clamps on both posts are opened - pushing down on the security catch unlocks and opens the clamp and securely fasten the top rail in place.



5

The system can be adjusted to a height of 94.5 inches (2.4 meters). That is just short of the 8 foot ceiling found in most homes.



6

The telescoping posts include height indicators that allow you to adjust both posts for a perfectly level top rail. A special one-way locking mechanism ensures that the post cannot retract on its own.



7

The next step in readying your partner for transfer is to position the sling around him and secure the sling straps to the lift*.



8

A BHM portable, battery-powered rechargeable lift* is used to gently lift and assist you in the safe and secure transfer of your loved one.

BHM Medical...because health matters

* Lift shown is sold separately and not included with Easytrack FS kit.