


# ErgoFit® Band sling

The ErgoFit® Band sling has a wide body component for increased support of the upper torso; minimizing the sling "riding up" under the arms of the person. Designed for use with BHM Medical sit and stand floor lifts, this sling is to be used for seat to seat transfers, toileting and pericare.

The ErgoFit® Band sling allows for removal of lower extremity clothing of the person without removal of the sling. It can also be used for practice weight bearing and balance for partially dependant people. Patient should have good upper body control to hold themselves up and lean back. The patient arms are positioned outside the Band sling at all times.

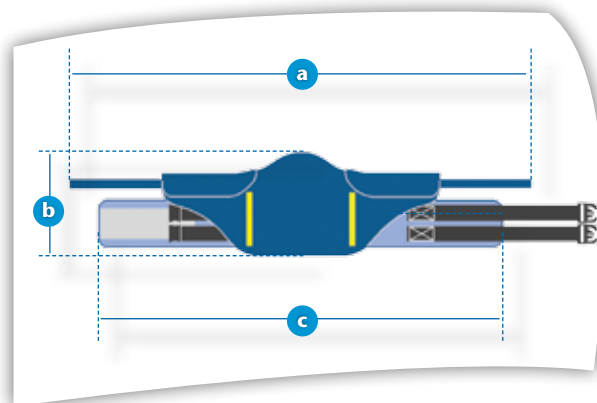

# Product	Description	Dimensions	
		in	cm
TST-S	Small Padded Band Sling	A	59 150
	Capacity	B	12 31
	45-100 lb / 20-45 kg / 3.15-7.14 st	C	43 109
TST-M	Medium Padded Band Sling	A	59 150
	Capacity	B	13 33
	100-200 lb / 45-90 kg / 7.14-14.29 st	C	48 122
TST-L	Large Padded Band Sling	A	62 158
	Capacity	B	14 37
	200-400 lb / 90-180 kg / 14.29-28.57 st	C	57 145
TST-XL	Extra Large Padded Band Sling	A	64 163
	Capacity	B	15 38
	400-600 lb / 180-272 kg / 28.57-42.86 st	C	66 168



**General Characteristics**

- Guarantee against manufacturing defects: 1 year
- Maximum loading capacity: 600 lb / 272 kg / 42.83 st
- Positioning handles at the back to help with transfers, to adequately position the user above a chair, to complete simple and quick transfers from a sitting to an inclined position and to help the user to swivel.
- Machine washable; easy to care for and clean.
- Strong nylon & polyester belts allowing the person to be in a sitting, semi-inclined or inclined position.
- User positioning system based on the color of the straps.
- Facilitate the change of underclothing, of incontinence towels as well as hygiene care of the patient.
- Belt with an abdominal band at the waist which enables the patient to stand up in confidence while being entirely supported. The band is easy to install.
- Additional soft and flexible padding to the armpits area, offering an additional protection and comfort.
- Dorsal support bands which distribute evenly the pressure.

**Technical Specifications**

Conform to ISO 10535 & CE marking

\*Conditionally to a normal use of an appropriate BHM Medical patient lift and maintenance in accordance with the labels.

