

proteX™ disinfectant spray

USE PROTEX™ DISINFECTANT SPRAY

- As a disinfectant on hard, nonporous surfaces
- For disinfecting of ultrasound transducers, probes, mammography compressor plates and other hard nonporous surfaces
- To clean and sanitize tables, chairs and countertops
- Use on nonporous athletic mats, wrestling mats, gymnastic mats, exercise equipment and training tables

Protex Disinfectant Spray is a one-step hospital use disinfectant, bactericidal according to the current AOAC Use-Dilution Test Method, Fungicidal according to The AOAC Fungicidal Test and Virucidal* according to the Virucidal qualification modified in the presence of 5% organic serum against:

(Bacteria) Acinetobacter baumannii • Brevibacterium ammoniagenes • Campylobacter jejuni • Community Associated Methicillin-Resistant Staphylococcus aureus–CA-MRSA • Enterobacter aerogenes • Enterococcus faecalis–Vancomycin resistant–VRE • ESBL Escherichia coli • Klebsiella pneumoniae • Legionella pneumophila • Pseudomonas aeruginosa • Pseudomonas cepacia • Salmonella choleraesuis • Salmonella schottmuelleri • Salmonella typhi • Serratia marcescens • Shigella dysenteriae • Staphylococcus aureus–Methicillin Resistant–MRSA • Staphylococcus aureus Multi-Drug Resistant • Staphylococcus aureus–Vancomycin Intermediate Resistant–VISA • Streptococcus pyogenes • Vibrio cholerae

(Viruses) Hepatitis B Virus (HBV)* • Hepatitis C Virus (HCV)* • Herpes Simplex Virus Type 1* • Herpes Simplex Virus Type 2* • HIV1 (AIDS Virus)* • Human Coronavirus* • Influenza Type A / Brazil* • Norwalk Virus* • Respiratory Syncytial Virus* • Rotavirus* • SARS Associated Coronavirus* • Vaccinia*

(Fungi) Trichophyton mentagrophytes • Aspergillus niger

USE IN:

- Hospitals
- Medical Related Facilities
- Nursing Homes
- Athletic Facilities
- Exercise Facilities

USE ON:

- Use Protex Disinfectant Spray on washable, hard, nonporous surfaces of: walls, counters, ultrasonic baths
- Other hard nonporous surfaces made of: laminated surfaces, metal, plastic, stainless steel, vinyl and plastic

For fungicidal activity: Protex Disinfectant Spray is an effective one-step fungicide against Trichophyton mentagrophytes when used on surfaces in areas such as locker rooms, dressing rooms, shower and bath areas and exercise facilities.

This product is not to be used as a terminal sterilant/high level disinfectant on any surface or instrument that (1) is introduced directly into the human body, either into or in contact with the bloodstream or normally sterile areas of the body, or (2) contacts intact mucous membranes but which does not ordinarily penetrate the blood barrier or otherwise enter normally sterile areas of the body.

KILLS HIV-1 AND HBV AND HCV ON PRE-CLEANED ENVIRONMENTAL SURFACES / OBJECTS PREVIOUSLY SOILED WITH BLOOD / BODY FLUIDS

in health care settings or other settings in which there is an expected likelihood of soiling of inanimate surfaces / objects with blood or body fluids and in which the surfaces/objects likely to be soiled with blood or body fluids can be associated with the potential for transmission of Human Immunodeficiency Virus Type 1 (HIV-1) (associated with AIDS) or Hepatitis B Virus (HBV) or Hepatitis C Virus (HCV).