

# QUICKVUE In-Line® Strep A TEST

## Test and Treat Today!™

Strep throat affects all age groups, but is most common in children between the ages of 5 and 15 years of age. Physicians diagnose strep throat based on symptoms, physical findings and culture.

QuickVue In-Line® Strep A is a truly one-step test, which allows for the rapid detection of group A streptococcal antigen, directly from patient throat swab specimens.

QuickVue In-Line Strep A's unique In-Line extraction method reduces hands-on time, and provides quick results allowing testing to take place right at the point of care. Extraction begins instantaneously and the two color test result gives clear readability.



### Rapid Diagnosis and Treatment

- Results in 5 minutes, or less
- Most patients tested can be diagnosed and treated in just one visit.

### Excellent Performance\*

- 92% sensitivity
- 99% specificity
- 96% overall agreement with SBA culture

### Easy to Use

- One easy step via In-Line extraction.
- Two-color endpoint for easy to read and interpret results.
- Single reagent method allows for less chance of operator error.

### Convenient Packaging

- All components necessary to perform a test are contained in the kit, including swabs.
- Room temperature storage (14 months from date of manufacture).
- Built-in Positive and Negative controls confirm the test has performed properly.
- Positive and Negative external controls provided in the kit.
- Individually packaged for convenient single-use testing and disposal.

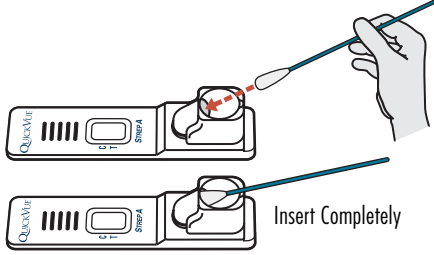
\*In-house study conducted at Quidel. Refer to Package Insert for additional performance claims.

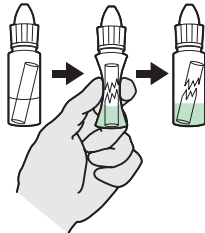
**QUIDEL**®  
CORPORATION


For more information on QuickVue In-Line Strep A test kits, visit our website or call your Quidel Account Manager at 1.800.874.1517 (U.S. only), or 858.552.1100 (outside U.S.).

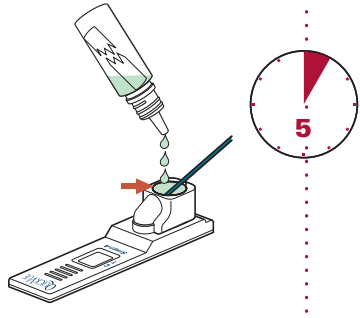
Quidel Corporation • Worldwide Headquarters • 10165 McKellar Court • San Diego, CA 92121 USA • [www.quidel.com](http://www.quidel.com)

## Procedure:

- 1**  
**MUST** use the swabs provided in the kit  


Insert Completely
- 2**  


Squeeze **ONCE** to break.
- 3**  


Shake
- 4**  


Quickly Fill to Rim



To order: Call your Quidel Account Manager at 1.800.874.1517

QuickVue In-Line® Strep A – 25 Test Kit:  
Catalog #00343

**QUIDEL**®  
CORPORATION

For more information on QuickVue In-Line Strep A test kits, visit our website or call your Quidel Account Manager at 1.800.874.1517 (U.S. only), or 858.552.1100 (outside U.S.).

Quidel Corporation • Worldwide Headquarters • 10165 McKellar Court • San Diego, CA 92121 USA • [www.quidel.com](http://www.quidel.com)

Distributor Information: